

*The vital energy for skin
Improving your skin's elasticity*

ULTRA TONING

- Dual-frequency cross-probing generates pressure changes that affect connective tissue and cell structure
- 4.0MHz RF intensive care for New Collagen and Elastin Generation




RF MODE



ULTRASOUND MODE(FACE)



ULTRASOUND MODE(BODY)

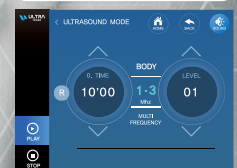

Head Office & Academy
13F 63, Ttukseom-ro 1-gil, Seongdong-gu, Seoul, Republic of Korea
Tel +82-2-1800-9095 Fax +82-70-5222-3741
www.filloshine.co.kr

FILLOSHINE
just science

ULTRA TONING



Product Name Ultra Toning

Item Name Ultrasonic Beauty Equipment

Intended Use Improvement of skin by generating multiple frequency-modulated ultrasonic waves

Principle of operation

1MHz/3MHz, 3MHz/10MHz, 4MHz(RF) Beauty equipment that improves skin by applying modulated ultrasonic waves

Ultrasonic Intensit

MAX 2.0 W/cm²+30% Beauty equipment that improves skin by applying modulated ultrasonic waves

Pulse / Pulse Continuous

1M/3M Pulse
3M/10M Pulse
1M Pulse Continuous
3M Pulse Continuous
10M Pulse Continuous


Probe(Hand piece)

One Probe - 1/3MHz (3.5cm²)
One Probe - 3/10MHz (1.5cm²)
One Probe - 4MHz (RF)

Principle of Ultra Toning


Improvement of skin by generating multiple frequency-modulated ultrasonic waves

1/3 MHz

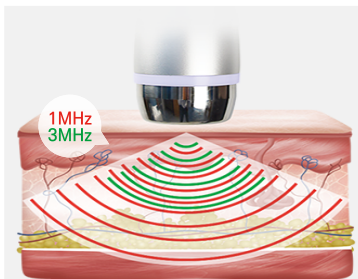

Penetration depth of deep dermal layer
BODY 1/3MHz : Dermis about 4.0-30mm

3/10 MHz

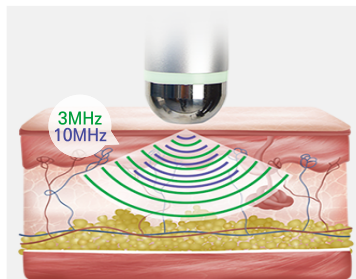

Penetration depth of epidermis and dermis
FACE 3/10MHz : Dermis about 0.5-4.0mm

Non-thermal(Physical) Effect

GAG (Hyaluronic Acid, Heparin, Dermatan)
Redistribution (redistribution) Cytokine secretion by ion gradient adjustment without damage to the dermis
Rhythmical Cell Massage (RCM) Effect


Thermal Effect

Collagen regeneration and synthesis

Biological Effect

Increased blood circulation due to vascular determination


4.0MHz RF intensive care for

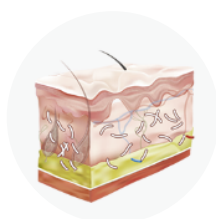

Collagen layer that loses elasticity in the dermis
creates wrinkles on the skin



The high frequency RF of 4MHz increases heat from 39°C to 45°C

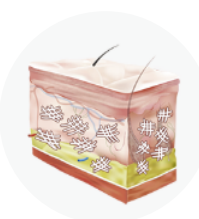

New Collagen and Elastin Generation

4.0MHz RF treatment increases the synergistic effect between 4MHz radio frequency after ultrasonic treatment, and due to the thermal energy effect of the dermis layer in the skin, it becomes collagen stimulation, wrinkle improving effect and up to 90 days by providing skin elasticity effect

Effects



Skin Brightening




Improvement of fine wrinkles




Promote regeneration after laser



Dissolving cellulite

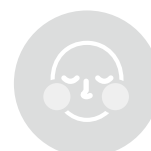

Reduction of facial redness

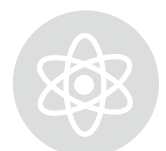

Promote collagen production